FRANÇAIS

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ENGLISH

REUSABLE FACE MASK FOR ADULTS AND CHILDREN

HOW TO PUT ON THE MASK

1. Before putting on the mask, wash hands with alcohol-based hand rub or soap and water.
2. Locate the nose clip to identify the top of the mask.
3. Place the mask over nose and mouth and place the elastics behind the ears.
4. Check that the mask covers the chin.
5. Adjust the nose clip to the shape of your nose.
6. Avoid touching the mask and your face while wearing the mask.

HOW TO REMOVE AND WASH THE MASK

1. For comfort and hygiene reasons, we recommend to not wear the mask for more than 8 hours.
2. Before removing the mask, wash hands with alcohol-based hand rub or soap and water.
3. Remove the mask without touching the front of the mask.
4. Store in a plastic bag if not at home.
5. Machine-wash the mask at 60°C with neutral soap after each use.
6. Do not tumble dry.

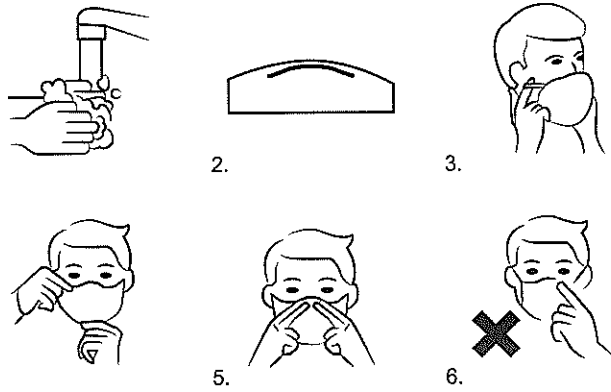
MATERIAL, MANUFACTURING AND CERTIFICATION

All our masks are Made in Europe using a water-repellent non-toxic polyester outer fabric and anti-bacterial cotton inside fabric. The masks have comfortable ear elastics and an adjustable nose clip.


Our masks follow production standards and are tested according to the European norms set for face masks level 3 as follows:

European Regulations for Level 3 face masks and test results after 5 washes	
Bacterial Filtration Efficiency (BFE)	- Following EN 14683:2019 European Regulations - Minimum required level 3 face masks BFE \geq 70% - Test results of our masks is 75%
Breathability	- Following EN ISO 9237:1995 European Regulations - Minimum required level 3 face masks Breathability \geq 8 l/min - Test results of our masks is 21 l/min

Laboratory tests of our masks are made by Citeve Laboratory, appointed by the Portuguese Government for test of face masks produced in Portugal.

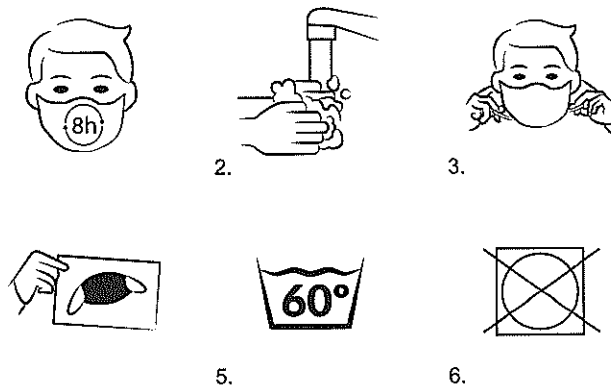Removing and washing the mask // Como retirar y lavar la mascarilla
 Ausziehen und waschen die Maske // Comment enlever et laver le masque
 Come rimuovere e lavare la mascherina // Afzetten en wassen van het masker
 Como colocar a máscara

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